



Juzo Expert & Strong

Product Guide

Compression Therapy with Juzo

For 100 years the Juzo family has lived by the simple philosophy that the garments we provide to Juzo dealers for your patients should be of the highest quality, therapeutic effectiveness and provide the utmost wearing comfort. But our commitment to excellence goes beyond manufacturing garments. We are passionate about what we do, and also believe in the importance of education and sharing knowledge, so we provide Juzo dealers with quality garments that are supported by knowledgeable, friendly and personal service.

- Family owned for 100 years, built on the philosophy that a medical garment should not inhibit a person's lifestyle.
- Juzo Academy – offers continuing education units and advanced knowledge of compression therapy (www.juzoacademy.com)
- Juzo E-learning – find helpful tools and learn more about compression therapy online (www.juzousa.com/dealer)
- Our sales and customer service team members are available to provide knowledgeable help with your compression therapy needs.

Made in USA of Imported Yarns

Our custom flat knit garments are made in the USA. This enables us to deliver quicker turnaround time to make sure your patients have the product when they need it.



Comfort

- Our garments are made using Fibersoft technology, an exclusive manufacturing process.
- Flat elastic and smooth seams, limits skin irritation and support mobility.
- Two-way stretch material increases mobility and comfort.

Ease of Use / Care

- Juzo Garments can be machine washed and tumbled dry.
- Covered compression threads (Fibersoft process) eases donning and doffing as product is smooth against skin.

Not Made With Natural Rubbers

- Juzo garments are made with synthetic elastomers.
- Resistant to oils, sweat, ointments and sunlight.
- Ideal for patients with latex allergies.

Appealing & Fit

- Juzo garments are available in a variety of colors, sizes and options, allowing you to customize for patient specific needs and style.
- Juzo's Fitting Guarantee on flat knit products assures that patients are comfortable with their garment for the perfect fit.

Therapeutic Compression

- An inlay thread, also known as a compression thread, is a circumferential thread that is knitted throughout the garment. By varying the tension of this thread, you can more precisely control the compression the stocking provides.
- Customized fit to individual measurements.

Available in the Following Colors:

Beige

Blue

Chestnut

Dark Blue

Gray

Fuchsia

Black

Violet

Indications for Compression Garments

Venous Disease Management

Chronic venous disease is a common disorder that affects the veins of the legs. Normally, these veins carry blood from the legs to the heart. Normal veins have a series of valves that open and close to direct blood flow from the legs up towards the heart. The valves also control the pressure in veins and capillaries in the legs.

Chronic venous insufficiency (CVI) is a condition caused by weak, worn or damaged valves in the veins of the legs. If the valves within the veins fail to work properly, blood can flow backwards and pool in the legs. The pooled blood can increase pressure in the veins. This can cause problems that are mild (leg heaviness, aching, dilated or unsightly veins) or severe (swelling, skin color changes, rashes on the legs, recurrent skin infections and chronic ulcers). People who develop these symptoms are said to have chronic venous insufficiency.

The primary role of compression garments in CVI is to assist and support the leg veins. Compression stockings help increase circulation in the supine position and manage leg swelling during prolonged sitting and standing.

Lymphedema Management


The primary role of compression garments and lymphedema management is to maintain the reduction of the swelling achieved during the intensive treatment phase of Complete Decongestive Therapy (CDT). Using a consistent level of compression is crucial in order to avoid re-accumulation of evacuated lymphedema fluid. Juzo attains consistency by providing high quality compression garments containing inlay threads, which are made of synthetic elastomers. These inlay threads are woven into the material in a continuous manner, which provides the correct level of graduated compression. It is important to realize that garments of lesser quality often do not contain inlay threads and may not be suitable for lymphedema management.

Burn Scar Management

Scars are developed by low-level burns and scalds, and are often treated with skin grafts. In order to manage the emergence of proliferating scars external compression is necessary to help close the wound. This way the scar tissue will heal quicker and scar thickness will be reduced. Compression garments also can be beneficial with existing scars.



Juzo Flat Knit Material Options

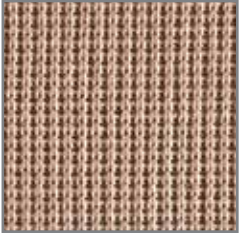

Juzo Expert Custom Garments

Seamed garment, extremely comfortable.

- Elastic
- Soft, comfortable material
- Dense knit for therapeutic containment
- UV Protection*

Expert: Juzo Expert has a dense knit yet is known for two way stretch elasticity and gentle material that provides maximum comfort and mobility. It's elasticity and softness makes it easier to don than most flat knit custom garments.

Expert Cotton: Juzo Expert Cotton has 100% cotton contact with the skin. Using only the best cotton fabric this garment is ideal for patients who have sensitive skin or those who prefer the feel and benefits of natural cotton.



Juzo Strong Custom Garments

Seamed garment with maximum containment and therapy.

- Short stretch properties
- Unique micro massage weave
- Maximum therapeutic effect
- UV protection*

Strong: Specialized, unique knit with high working pressure and micro massaging effect on the tissue. Juzo Strong is ideal for patients with advanced cases of lymphedema and fibrosis that are difficult to treat.



Juzo Silver Custom Garments

Safe, natural permanently bonded pure silver by X-Static.

- Antimicrobial fabric protection
- Heat dissipating
- Odor reduction
- Anti-static

Juzo Silver Strong and Expert: Maintaining healthy skin is an important part of managing lymphedema, burns and venous ulcers. Juzo Silver flat knit custom garments offer an antimicrobial solution to manage harmful bacteria that can lead to infections or skin breakdown.

- **Lymphedema patients** with cuts, scratches and cracks in the skin that may allow bacteria to enter the body and lead to cellulitis or lymphangitis.
- **Venous disease patient** can develop underlying problems such as, infections, swelling, skin irritations and ulcers.
- **Burn patients with scalds and scars** are highly susceptible to bacterial infections.

* Juzo Expert & Strong has UV Standard of 80. This was tested and certified by Hohenstein Institute.

Custom Flat Knit Lower Extremity

Juzo flat-knit lower extremity garments with seam are recommended as part of the management process in conjunction with other therapies. Customized to any shape or size, the breathable, durable elastic material is comfortable and wearable so the patient is able to engage in everyday activities while receiving the therapeutic benefits of compression.

FiberSoft Technology

- Extremely comfortable & durable
- Resistant to most lotions, ointments and creams.

Not Made Using Natural Rubbers

- No latex allergies
- Machine wash and dry

Over 100 Years Of Knitting Experience

- Flat and non-irritating seams
- Elastic knit product easy to don and doff and moves with patients for natural mobility
- Anatomically shaped
- Fitting guarantee

Micro Massage Effect (Juzo Strong and Strong Silver)

- Enhance lymph flow to help manage and maintain limb circumference
- Soften hard fibrotic tissue

Excellent Containment

- Dense knit
- Knitted to provide excellent containment throughout the garment.



Custom Flat Knit Lower Extremity



Compression Therapy for Leg and Foot

Venous / Lymphedema Disease: compression stockings and pantyhose are used in the management of assisting the venous and lymphatic system and minimizing edema. Patients with edema in forefoot and toes area should use foot portions.


Burn Therapy: compression stockings and pantyhose are used to apply pressure to the surface of the scar to heal quicker and protect the scar area while also reducing edema.

Stockings and Pantyhose

Designed to be comfortable, breathable and an aesthetically appealing garment for patients allowing them to return to normal daily activities. Juzo compression stockings and pantyhose are available custom made in Juzo Expert, Expert Cotton, Expert Silver, Strong and Strong Silver.




Custom Flat Knit Lower Extremity

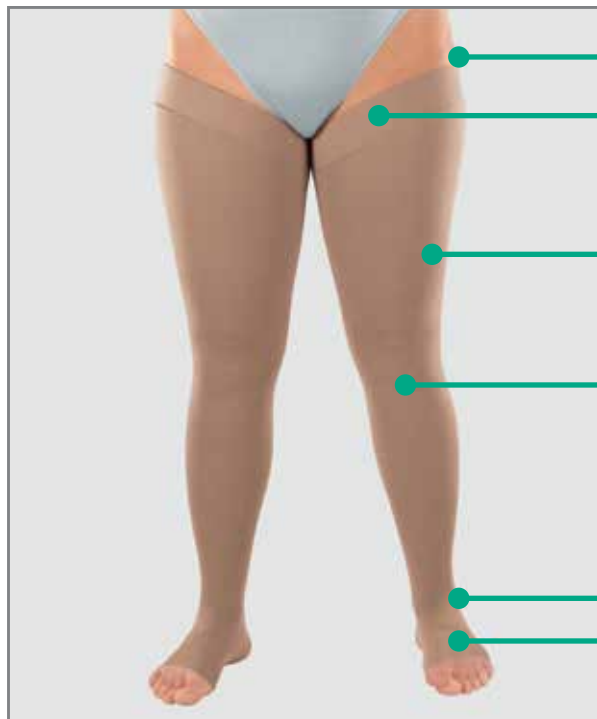

Thigh high stocking AG

- Slant top at "G"
- Beaded silicone band
- Silicone stoppers
- Anatomically shaped
- Pressure pad
- Slant toe

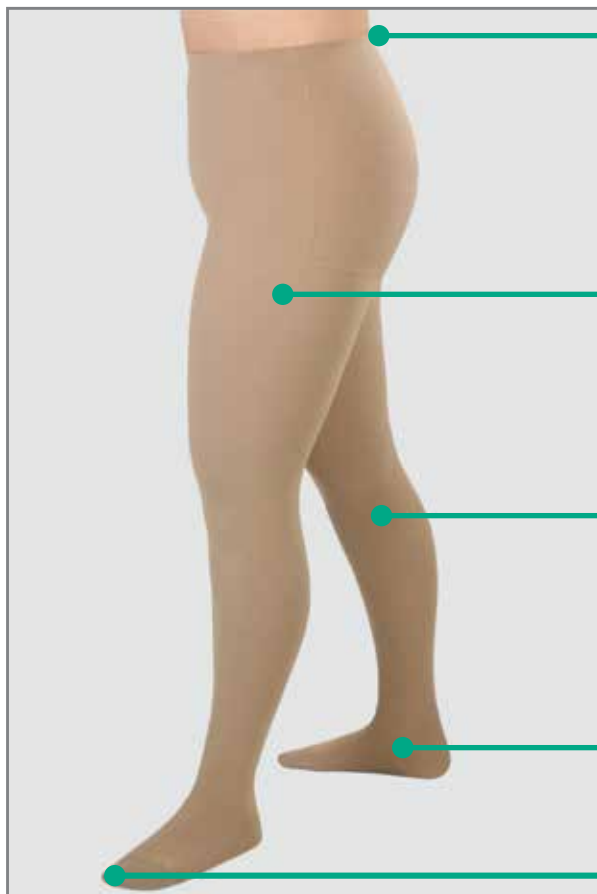

Pantyhose AT

- Adjustable waistband
- Silicone stoppers (lateral side)
- Anatomically shaped
- Pressure pad
- Full foot

Custom Flat Knit Lower Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|---|---|
|  | <p>Slant Toe Styles: AD/AG/AT</p> | <ul style="list-style-type: none"> • Slanted at metatarsals • Compression up to the toes • Ideal for edema treatment in combination with toe portion (see picture) • Optimal fit/wearing comfort • Edema therapy | <p>T01 - Closed T03 - Slant</p> |
|  | <p>VY Heel Styles: AD/AG/AT</p> <ul style="list-style-type: none"> • Heel dart | <ul style="list-style-type: none"> • Wrinkling in arch area is reduced • Improved fit for extreme arch edema • Wheelchair patients • Edema therapy | <p>D02 - VY Heel</p> |
|  | <p>Porosa Reduced compression and increased elasticity beginning at point D</p> <p>Style: AG Optional with Expert, Expert Cotton, Expert Silver</p> | <ul style="list-style-type: none"> • Improved wearing comfort in popliteal region with higher compression classes • Phlebology | |
|  | <p>Full Knit Continuous knit throughout the stocking.</p> <p>Style: AG Standard with Expert, Expert Cotton, Expert Silver, Strong, Strong Silver</p> | <ul style="list-style-type: none"> • Higher remaining pressure on thigh • Edema therapy • Skin therapy | <p>Standard</p> |
|  | <p>Slant Top Diagonal finish at D or G</p> <p>Style: AD/AG</p> <p>Important: measure exact inside lengths to D or G Outside length not needed</p> | <ul style="list-style-type: none"> • Optimized compression reduction (proximal) • Reduces bulging tissue at the top of the leg • Improved fit on larger legs • Prevents constriction at the top of the garment • Adipose tissue • Edema therapy | <p>T10 - Slanted T11 - Slanted Max</p> |

Custom Flat Knit Lower Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|---|--|
|  | <p>Beaded Silicone Border</p> <p>Style: AD/AG</p> <ul style="list-style-type: none"> • AD Standard 3.5 cm - Optional 5 cm • AG Standard 5 cm | <ul style="list-style-type: none"> • Prevents slipping • Inconspicuous • Very good fixation in low-hair areas • With AD: For securing a tapered leg shape • With AG: Alternative to the hip attachment or AT | <p>S01 - 3.5 cm S10 - 5 cm</p> |
|  | <p>Silicone Stopper</p> <p>Style: AD, AG, AT</p> <p>Positioning of silicone band stoppers:</p> <ul style="list-style-type: none"> • Medial • Lateral • Anterior • Posterior | <ul style="list-style-type: none"> • For even better adhesion • Tapered thigh and/or tapered leg shape <p>Note: (AG) A maximum of three stoppers are possible per leg. (AD) A maximum of two.</p> | <p>S30 - Anterior S31 - Posterior S32 - Medial S33 - Lateral</p> |
|  | <p>Silicone Border Sewn Inside</p> <p>Style: AD/AG</p> <ul style="list-style-type: none"> • Only with slanted top • Full or 3/4 available | <ul style="list-style-type: none"> • Enhanced stability and adhesion | <p>3/4 Silicone Border S02 - 3.5 cm inside (AD) S11 - 5 cm inside</p> <p>Full Silicone Border S03 - 3.5 cm inside (AD) S12 - 5 cm inside</p> |
|  | <p>Silicone Stopper Inner Thigh</p> <p>Style: AG</p> | <ul style="list-style-type: none"> • Holds thigh high in place, prevents rolling • Used with a body part or hip attachment | <p>S13 - 5 cm, inner thigh</p> |

Custom Flat Knit Lower Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|---|-----------------------------------|
|  | <p>Body Part</p> <p>Style: KT Use with style: AG</p> <ul style="list-style-type: none"> • Silicone border on inner thigh available • 18-21 or 23-32 compression in abdominal region • With or without hook and loop closure in the front | <ul style="list-style-type: none"> • Patients in need of compression in the abdomen • Alternative for patients with sensitive skin • Securely keeps stocking(s) in place • Edema therapy • Scar therapy • Phlebology | |
|  | <p>Hip Attachment</p> <p>Non compressive hip panel with waist belt</p> <p>Use with style: AG</p> <ul style="list-style-type: none"> • Silicone border available inner thigh • Full silicone border available | <ul style="list-style-type: none"> • Alternative for patients with sensitive skin • More stability on the outside of the thigh • Edema therapy • Scar therapy • Phlebology | <p>H01 - Right H02 - Left</p> |
|  | <p>Hip Attachment Sewn as One (Chap Style)</p> <p>Bilateral stockings connected to a single waist belt</p> <p>Use with style: AG</p> | <ul style="list-style-type: none"> • Optimal fit • Alternative to open crotch pantyhose • Edema therapy • Scar therapy • Phlebology | <p>H05 - Sewn as One</p> |
|  | <p>Knee Dart</p> <p>Use with style: AG, AT</p> <p>Important: measure exact point "e" (floor to mid patella)</p> | <ul style="list-style-type: none"> • Reduces creases in the knee area (sensitive knee area) • Optimized fit in knee area • Wheelchair patients • Not necessary in expert material • Movement restrictions • Edema therapy • Scar therapy | <p>DO1 - Knee Dart</p> |






Custom Flat Knit Lower Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|---|---|--|
|  | <p>Stocking With Comfort Patch Styles: AG/AT</p> <ul style="list-style-type: none"> • Smooth liner available • Silver liner available, anti odor • Custom sizing and location available | <ul style="list-style-type: none"> • Reduces irritation in the knee hollow in cases of sensitive skin • Very soft, with flat seam • Silver for inflamed or sensitive skin • Edema therapy • Wheelchair patients | <p>P20 - Smooth Knee Patch P30 - Silver Smooth Knee Patch</p> |
|  | <p>Stocking With Comfort Patch at Ankle Crease Styles: AD/AG/AT</p> <ul style="list-style-type: none"> • Smooth liner available • Silver liner available, anti odor • Custom sizing and location available | <ul style="list-style-type: none"> • Reduces irritation at the base of the ankle / top of the foot, in cases of sensitive skin • Very soft, with flat seam • Silver for inflamed or sensitive skin • Edema therapy • Wheelchair patients | <p>P21 - Smooth Patch P31 - Smooth Silver Patch</p> |
|  | <p>Pressure Pad Style: AD/AG/AT Model # 9400</p> <ul style="list-style-type: none"> • Can be positioned medial lateral | <ul style="list-style-type: none"> • For increased compression in the concave areas around the malleolis • Suited for venous ulcers • Edema therapy | |
|  | <p>Silver Liner Style: AD Model # 9640</p> <ul style="list-style-type: none"> • Made with silver fibers • Mild, non-graduated compression | <ul style="list-style-type: none"> • Can be used as a liner under a flat or circular knit stocking • Ideal for the integration of wound dressings for the treatment of small skin wounds • Phlebology therapy • Edema therapy | |

Custom Flat Knit Lower Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES | | | | | | | | | | | | | | | | | | |
|--|---|---|--------------------------------|-------|-----|-----|-----|---|---|---|---|---|---|---|---|---|---|---|---|--|--|
|  | <p>Panty Compression Standard combinations:</p> <table border="1"> <thead> <tr> <th>AT</th> <th>Leg</th> <th>Panty</th> </tr> </thead> <tbody> <tr> <td>CCL</td> <td>CCL</td> <td>CCL</td> </tr> <tr> <td>1</td> <td>1</td> <td>1</td> </tr> <tr> <td>2</td> <td>2</td> <td>2</td> </tr> <tr> <td>3</td> <td>3</td> <td>2</td> </tr> <tr> <td>4</td> <td>4</td> <td>2</td> </tr> </tbody> </table> <p>Any other combination possible</p> | AT | Leg | Panty | CCL | CCL | CCL | 1 | 1 | 1 | 2 | 2 | 2 | 3 | 3 | 2 | 4 | 4 | 2 | <ul style="list-style-type: none"> Compressive Juzo body garments guarantee an optimal fit Edema therapy and scar therapy in hip / abdomen / buttocks area | |
| AT | Leg | Panty | | | | | | | | | | | | | | | | | | | |
| CCL | CCL | CCL | | | | | | | | | | | | | | | | | | | |
| 1 | 1 | 1 | | | | | | | | | | | | | | | | | | | |
| 2 | 2 | 2 | | | | | | | | | | | | | | | | | | | |
| 3 | 3 | 2 | | | | | | | | | | | | | | | | | | | |
| 4 | 4 | 2 | | | | | | | | | | | | | | | | | | | |
|  | <p>Panty with Gluteal Extension (optional)</p> <p>Rear panty panel extending beyond the gluteal fold</p> | <ul style="list-style-type: none"> Recommended for very large circumference measurements in hip, abdominal / buttocks area Edema therapy and scar therapy in hip/abdomen/buttocks area | 1ATG01 | | | | | | | | | | | | | | | | | | |
|  | <p>Panty Without Waistband</p> | <ul style="list-style-type: none"> Best for children and adults with a slim fit / thin waist Inconspicuous flat fabric seal from the waistline With children and adults with a thin figure Edema therapy Scar therapy | Standard | | | | | | | | | | | | | | | | | | |
|  | <p>Panty With Adjustable Waistband</p> <ul style="list-style-type: none"> Individual adjustment possibility with the waistband | <ul style="list-style-type: none"> Reduces slipping when bending or sitting Large circumference measurements for the hips & waist Greater adjustability Edema therapy Scar therapy | W01 - Adjustable Waistband | | | | | | | | | | | | | | | | | | |
|  | <p>Panty With Elastic Waistband</p> <p>Non-adjustable waist belt</p> | <ul style="list-style-type: none"> Visually flat and elastic seal No folding of the fabric band Moderate circumference for the hips and waist Lies flat, prevents rolling Edema therapy Scar therapy Lipedema (with waist) | W02 - Non-Adjustable Waistband | | | | | | | | | | | | | | | | | | |

Custom Flat Knit Lower Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|---|--|---|
|  | <p>Panty With Vertical Velcro Closure</p> <ul style="list-style-type: none"> • Front or sides – also possible with zipper | <ul style="list-style-type: none"> • For extremely narrow waist and very wide hips • Diapers • Edema therapy • Scar therapy | <p>F22 - Velcro Fastener Z01 - Zipper</p> |
|  | <p>Leggings</p> <p>Style: BT</p> | <ul style="list-style-type: none"> • Easier to put on when one has restricted movement or when treating extreme body dimensions • Can be used in combination with other garments • Edema therapy • Scar therapy | |
|  | <p>Capri Stockings</p> <p>Style: BT</p> | <ul style="list-style-type: none"> • Easier to put on when one has restricted movement capability or when treating extreme body dimensions • Can be used in combination with other garments • Edema therapy • Scar therapy | |
|  | <p>Biker Shorts</p> <p>Style: FT</p> <ul style="list-style-type: none"> • Leg extension of at least 15 cm recommended with adults | <ul style="list-style-type: none"> • Easier to put on when one has restricted movement capability or when treating extreme body dimensions • Can be used in combination with other garments • Edema therapy • Scar therapy | |





Custom Flat Knit Lower Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|---|--|---|
|  | <p>Pantyhose With Leg Extension</p> <ul style="list-style-type: none"> Specify left or right leg <p>Note: leg extension should be at least 15 cm</p> | <ul style="list-style-type: none"> Targeted compression with edemas or scars of the affected body region Edema therapy Scar therapy | <p>Please specify which leg</p> <p>1ATL01 - Right Leg 1ATL02 - Left Leg</p> |
|  | <p>Panty With Crotch Panel (Standard)</p> | <ul style="list-style-type: none"> Edema therapy | <p>Standard</p> |
|  | <p>Panty With Mesh Crotch Panel</p> | <ul style="list-style-type: none"> Greater air circulation Edema therapy Scar therapy | <p>C02 - Mesh Panel C03 - Mesh Panel Silver</p> |
|  | <p>Panty With Open Crotch</p> <ul style="list-style-type: none"> Not recommended for larger circumferences (loss of compression) | <ul style="list-style-type: none"> Greater air circulation Added convenience Edema therapy Scar therapy | <p>C01 - Open Crotch</p> |

Custom Flat Knit Lower Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|---|---|
|  | <p>Pantyhose With Men's Gusset</p> <p>Style: AT</p> | <ul style="list-style-type: none"> • Ideal fit in the genital area • For men • Edema therapy • Scar therapy | <p>C04 - Gussett C05 - Silver Gussett</p> |
|  | <p>Pantyhose With Fly</p> <p>Style: AT</p> | <ul style="list-style-type: none"> • Added convenience • Scar therapy • Muscle injuries | <p>F10 - Pantyhose with Fly</p> |
|  | <p>Calf Legging Segment</p> <p>Style: BD</p> | <ul style="list-style-type: none"> • Scar therapy • Muscle injuries | |
|  | <p>Thigh Legging Segment</p> <p>Style: EG</p> | <ul style="list-style-type: none"> • Scar therapy • Muscle injuries | |

Custom Flat Knit Lower Extremity






Foot Portion

Designed to meet the contour and unique anatomy of a patient's foot while providing breathability and comfort. Juzo compression foot portions are available custom made in Juzo Expert, Expert Cotton, Expert Silver, Strong and Strong Silver.



Foot portion with open toes

Flat, elastic seams

| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|---|---------------------------|
|  | Foot Portion Style: FP | <ul style="list-style-type: none"> • More compression with distinct arch and toe edema • Edema therapy • Burn scar therapy | |
|  | Foot Portion With Open Fifth Metatarsal Standard: slanted opening • Straight opening also possible upon request | <ul style="list-style-type: none"> • Individual treatment possibilities on the pressure-sensitive small toes • Edema therapy • Burn scar therapy | T06 - Open 5th Metatarsal |
|  | Foot Portion With Closed Toes Style: FP | <ul style="list-style-type: none"> • Compression on toe caps • Edema therapy • Burn scar therapy • After amputation | T01 - Closed Toe |

Custom Flat Knit Upper Extremity

Juzo flat knit upper extremity garments with seam are recommended as part of the management process in conjunction with other therapies. Customized to any shape or size, Juzo flat knit garments with seams work hard to contain any swelling that may occur. Some styles are available with cotton or silver fiber, for patients with sensitive skin, allowing patient to stay comfortable in any activity or climate.

FiberSoft Technology

- Extremely comfortable & durable
- Resistant to lotions, ointments and creams

Not Made with Natural Rubber

- No latex allergies
- Machine wash and dry

Over 100 Years of Knitting Experience

- Flat and non-irritating seams
- Elastic knit product easy to don and doff and moves as patient moves
- Anatomically shaped
- Fit guarantee

Micro Massage Effect (Juzo Strong and Strong Silver)


- Increase lymph flow to help manage extremity arm / hand circumference
- Helps break down hard, fibrotic tissue

Excellent Containment

- Dense knit
- Knitted to provide excellent containment throughout the garment.



Custom Flat Knit Upper Extremity

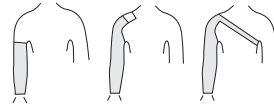


Compression Garments for Arm and Hand

Lymphedema Therapy: compression is an important tool in the management of arm / hand swelling due to primary and secondary lymphedema. Compression gloves and gauntlet help manage lymph fluid from accumulating in hand and fingers and encourage flow from the tissue.


Burn Therapy: compression is used to eliminate bothersome irritation and apply surface pressure to help heal and protect the scar area while also reducing edema.

Arm Sleeve

Designed fine knit and soft fabric offer comfort and support during daily activities. Juzo compression flat knit arm sleeves with seam are available custom in Juzo Expert, Expert Cotton, Expert Silver, Strong and Strong Silver.

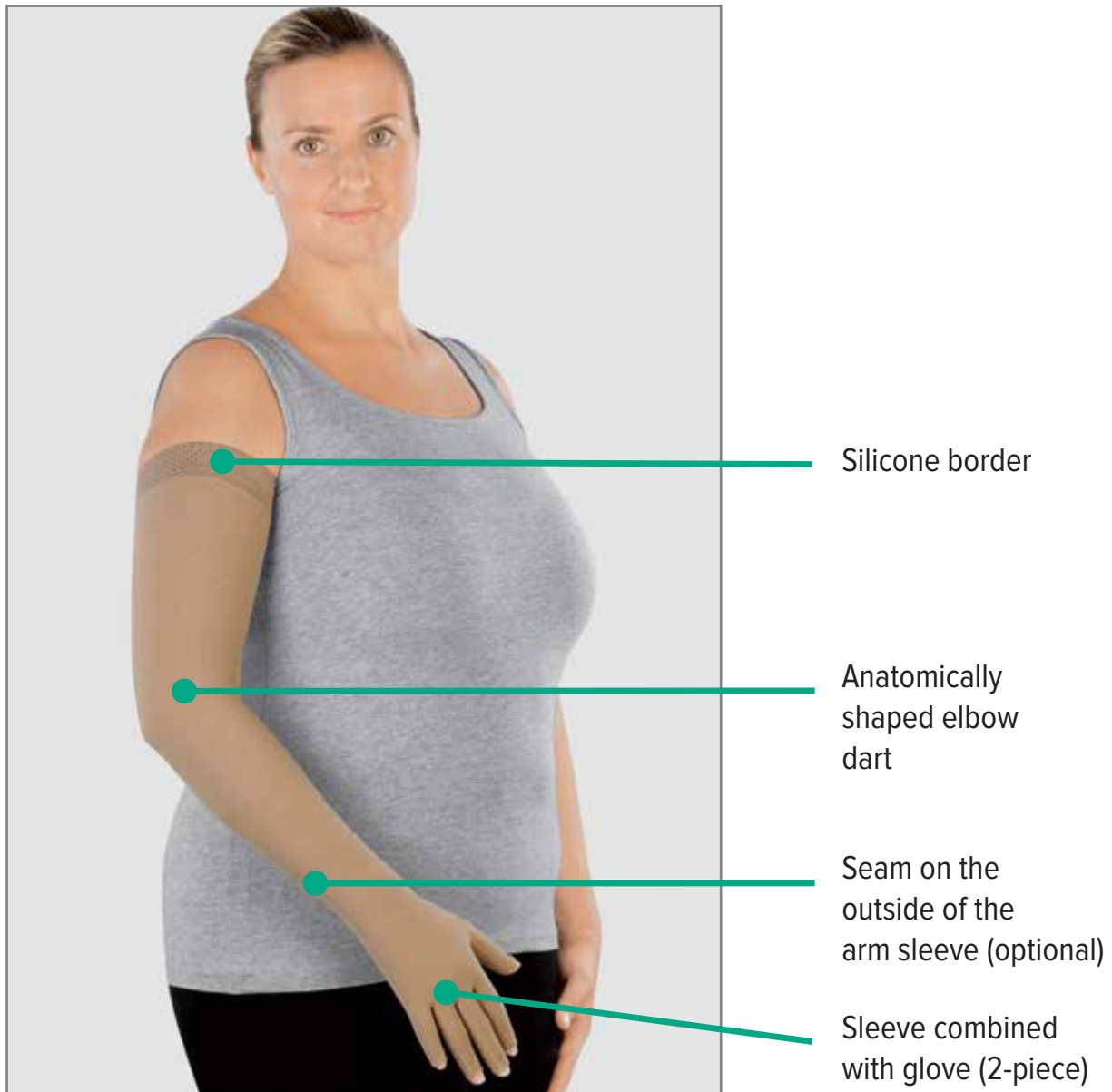


Custom Flat Knit Upper Extremity

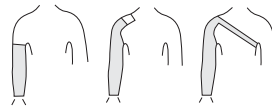

| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|---|---|--|
|  | <p>Arm Sleeve With a Silicone Band At “G”</p> <p>Style: CG</p> | <ul style="list-style-type: none"> • Securely keeps sleeve in place • Inconspicuous | <p>S01 - 3.5 cm Silicone Border</p> <p>S10 - 5 cm Silicone Border</p> |
|  | <p>Arm Sleeve With a Silicone Stopper</p> <p>Style CG</p> <p>5x5 cm</p> <p>No more than 2</p> | <ul style="list-style-type: none"> • Securely keeps sleeve in place • Max of two | <p>S30 - Anterior</p> <p>S31 - Posterior</p> <p>S32 - Medial</p> <p>S33 - Lateral</p> |
|  | <p>Arm Sleeve With a Silicone Band, sewn inside</p> <p>Style CG</p> <ul style="list-style-type: none"> • Used only in connection with a slanted top | <ul style="list-style-type: none"> • Increased stability and adhesion with very conical shapes • Edema therapy | <p>S03 - 3.5 cm Silicone Border Sewn on the Inside</p> <p>S12 - 5 cm Silicone Border on the Inside</p> |
|  | <p>Arm Sleeve With a 3/4 Silicone Band, Sewn Inside</p> <p>Style: CG</p> <ul style="list-style-type: none"> • For sensitive extremities, ¾ inside silicone band is also available | <ul style="list-style-type: none"> • Increased stability and adhesion with very conical shapes • Edema therapy • Scar therapy | <p>S02 - 3/4 Silicone Sewn Inside</p> <p>S11 - 5 cm 3/4 Silicone Sewn Inside</p> |
|  | <p>Arm Sleeve With Shoulder Strap</p> <p>Style: CH</p> | <ul style="list-style-type: none"> • Alternative to silicone border • Securely keeps the sleeve in place • Edema therapy • Scar therapy | |

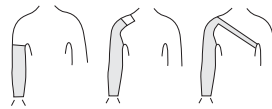

Custom Flat Knit Upper Extremity




| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|---------|---|---|---|
| | <p>Elbow Dart Style: CG, CH</p> <ul style="list-style-type: none"> Standard with Strong, Strong Silver Typically not required for Expert, Expert Cotton or Expert Silver <p>Important: measure exact length wrist to middle of elbow (c-e)</p> | <ul style="list-style-type: none"> Reduces creasing in elbow Optimal fit Sensitive elbow area Edema therapy Scar therapy Improves mobility | <p>D01 - Dart at Point "E"</p> |
| | <p>Arm Sleeve With Comfort Patch at "E"</p> <p>Style: CG, CH</p> <ul style="list-style-type: none"> Smooth liner available Silver liner available, anti-odor Custom sizing and location available | <ul style="list-style-type: none"> Reduces irritation at the bend of the elbow / inner elbow Very soft with flat seam Silver for inflamed or sensitive skin Edema therapy Scar therapy | <p>P20 - Smooth Patch P30 - Smooth Silver Patch P31 - Smooth Silver Patch Custom Made</p> |
| | <p>Arm Sleeve With Slant Top at "G"</p> <p>Style: CG</p> <p>Important: measure exact length CG (inside of arm, outside length not required)</p> | <ul style="list-style-type: none"> Reduces excess tissue protrusion (outer and inner arm) Optimal compression at the top of the sleeve Edema therapy | <p>T10 - Slanted Top T11 - Slanted Top Max</p> |
| | <p>Arm Sleeve With Seam On the Outside of the Arm</p> <p>Style: CG</p> <ul style="list-style-type: none"> With elbow dart only | <ul style="list-style-type: none"> With sensitive skin in the inner elbow Edema therapy Scar therapy | <p>S20 - Seam on the Outside (lateral)</p> |

Custom Flat Knit Upper Extremity




| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|---|---|--|-------------------------------|
|  | <p>Arm Sleeve With a Velcro to Bra Attachment</p> <p>Style: CH</p> <ul style="list-style-type: none"> • To be fixed to the bra strap • No shoulder compression | <ul style="list-style-type: none"> • Alternative to silicone band • Edema therapy • Scar therapy | <p>F20 - Velcro to Bra</p> |
|  | <p>Arm Sleeve With Bolero Connection</p> <p>Style: CH</p> <ul style="list-style-type: none"> • Requires patient mobility • No compression in the shoulder / back area. | <ul style="list-style-type: none"> • Securely keep the sleeve in place • Alternative to silicone border • For bilateral management • Edema therapy • Scar therapy <p>Note: requires patient mobility</p> | <p>S25- Bolero Attachment</p> |

Custom Flat Knit Upper Extremity

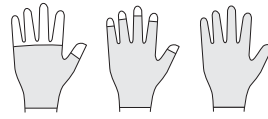


Glove

Designed to fit the complex anatomy of a patient's hand, while also providing comfort and durable wear for everyday activities. Juzo compression gloves are available custom in Juzo Expert, Expert Cotton, Expert Silver, Strong and Strong Silver.

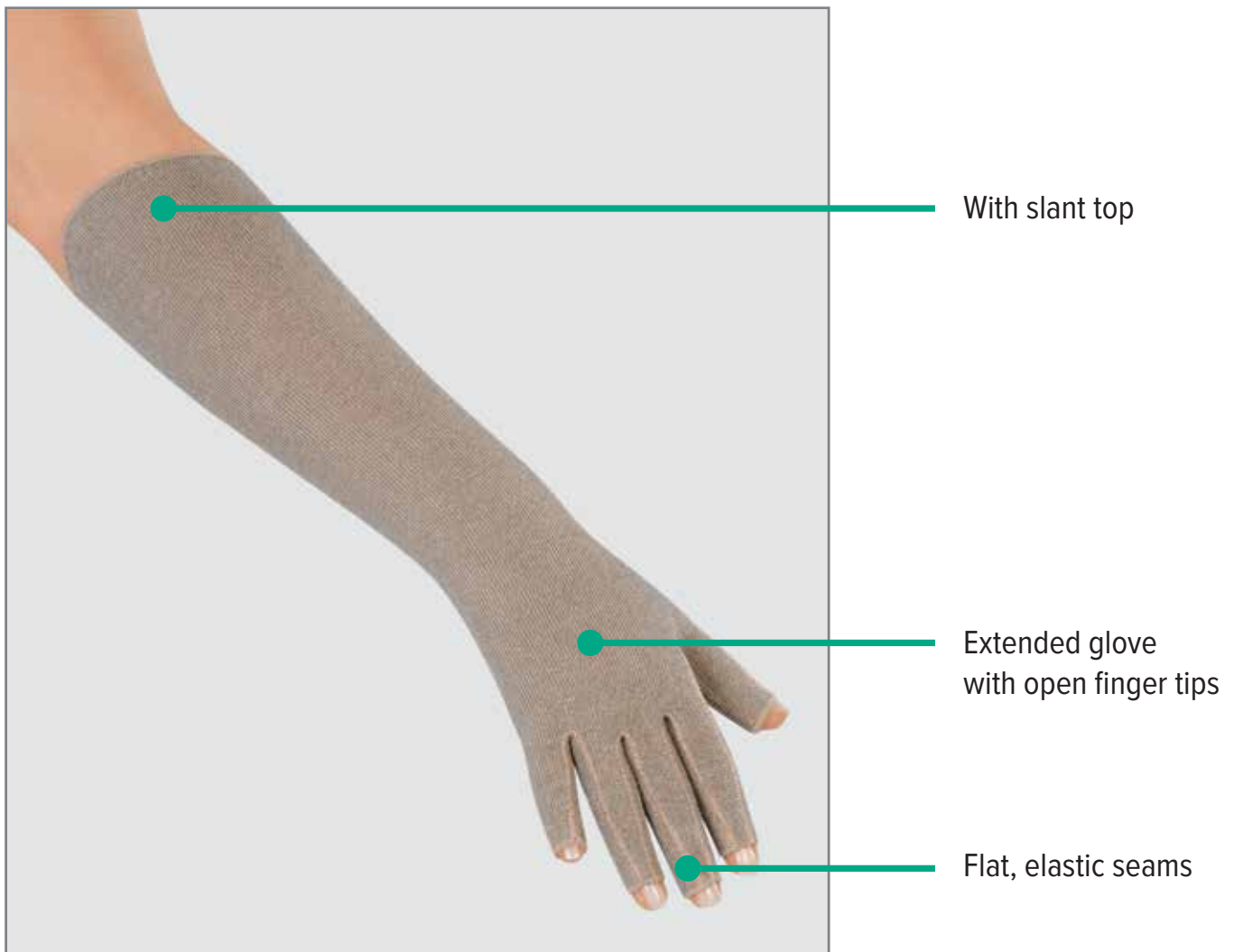




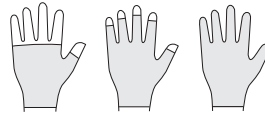

Custom Flat Knit Upper Extremity

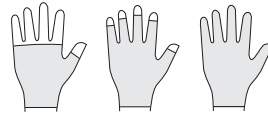


| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|---|--|---|
|  | <p>Glove With Open or Closed Fingers</p> <p>Style: ACFS</p> <ul style="list-style-type: none"> • Standard 6 cm wrist extension | <ul style="list-style-type: none"> • Best possible fit • Optimal wearing comfort • Edema therapy • Scar therapy | <p>F30 - Closed Fingers (specify)</p> |
|  | <p>Glove With Wrist Extension</p> <ul style="list-style-type: none"> • Extended length up to mid to lower arm (up to 20 cms) • Possible with silicone band and/or slant top | <ul style="list-style-type: none"> • With edemas or scars in hand and lower arm area • Edema therapy • Scar therapy | <p>E01 - Wrist Extension</p> |
|  | <p>Pressure Pad for the Hand</p> <p>Item #9420</p> <ul style="list-style-type: none"> • Optional sewn in • Targeted pressure increase | <ul style="list-style-type: none"> • Removable pressure pad • Best possible fit • Optimal wearing comfort • Edema therapy • Scar therapy | |
|  | <p>Glove With Pressure Pad Pocket</p> <ul style="list-style-type: none"> • Pad removable • Targeted pressure increase • Regular or elongated pad | <ul style="list-style-type: none"> • Helps stabilize removable pad • Best possible fit • Optimal wearing comfort • Edema therapy • Scar therapy | <p>P11 - Dorsal side P12 - Palm side</p> |
|  <p><i>glove inside out to show silver comfort patch</i></p> | <p>Comfort Patch For Thumb Web</p> <ul style="list-style-type: none"> • Smooth liner available • Silver liner available, anti odor, anti microbial • Custom sizing and location available | <ul style="list-style-type: none"> • Reduces irritation in thumb web in cases of sensitive skin • Edema therapy • Scar therapy | <p>P22 - Smooth Patch Thumb Web P32 - Smooth Patch Silver Thumb Web</p> |

Custom Flat Knit Upper Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|---|--|---------------------|
|  | <p>Gauntlet With Thumb Stub</p> <p>Style: AC</p> | <ul style="list-style-type: none"> • Simple treatment in hand area • Scar therapy | |
|  | <p>Finger Segment</p> <p>Style: F</p> <ul style="list-style-type: none"> • Open or closed | <ul style="list-style-type: none"> • For the treatment of individual fingers • Scar therapy | |
|  | <p>Sleeve and Glove, One Piece</p> <p>Styles: ACG, ACH (with strap)</p> <ul style="list-style-type: none"> • With slant top and/or silicone band (if requested) | <ul style="list-style-type: none"> • No overlapping in the wrist as with two piece treatment • Edema therapy • Swelling extending over the hand joint • Scar therapy | |
|  | <p>Glove With Zipper</p> <p>Style: ACFS</p> <ul style="list-style-type: none"> • Individual zipper positioning possible | <ul style="list-style-type: none"> • If skin is damaged • Edema therapy • Scar therapy | <p>Z01 - Zipper</p> |

Custom Flat Knit Upper Extremity

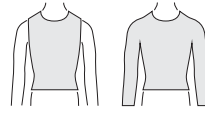


Compression Therapy for Upper Body

Lymphedema Therapy: Often treating only the arm with compression is not sufficient. Truncal lymphedema, which affects the chest region, frequently develops following breast or lung cancer treatment and can be present with or without significant involvement of the adjacent arm.

Burn Therapy: With large scars i.e. burns and scalds in the thorax area, Juzo offers garments for special needs such as compression body suits used by women and children or in combination with stockings.

Thorax Garments

Unique features which enable an ease of use for every day wear and challenges. The garments have great wearing properties such as breathable and soft knit designed for optimal wearing comfort. Juzo uses only flat-knit fabrics Juzo Expert and Juzo Expert Silver in the manufacturing of thorax garments.

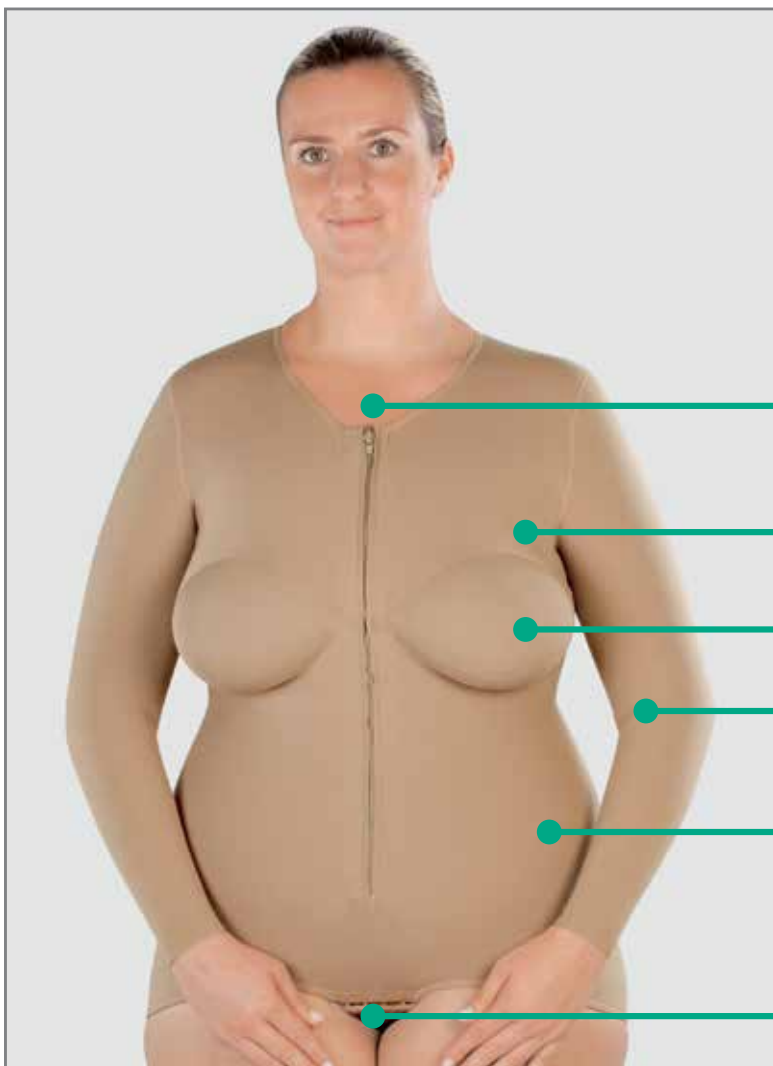

* Mesh area in the axilla ensures optimal air circulation

Zipper

Mesh area in the axilla (standard)*


Seamless breast cups





Anatomically shaped elbow darts (optional)

Ends at the waist (Standard)

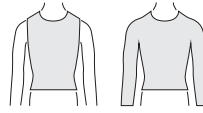
Optional crotch panel and fastener



Custom Flat Knit Upper Extremity




| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|--|---|
|  | <p>Thorax Garment With Arm Extension(S)</p> <p>Style: THSG</p> <ul style="list-style-type: none"> • Can be combined with long or short sleeve (s) | <ul style="list-style-type: none"> • Optimal compression of the shoulder and upper body area • Edema therapy • Scar therapy | <p>Please specify “with extension” and which Arm (left / right)</p> |
|  | <p>Thorax Garment With Crotch Panel and Fastener</p> <ul style="list-style-type: none"> • Appropriate for children and women | <ul style="list-style-type: none"> • Prevents sliding of the thorax bandage • Edema therapy • Scar therapy | <p>Please specify crotch panel with fastener</p> |
|  | <p>Thorax Garment With A Sewn-in Crotch Panel</p> | <ul style="list-style-type: none"> • Prevents sliding of the thorax bandage • Scar therapy | <p>Please specify sewn-in crotch panel</p> |
|  | <p>Thorax Garment With Crotch Panel and Velcro</p> <ul style="list-style-type: none"> • Velcro is removable, for individual fastening | <ul style="list-style-type: none"> • Velcro already sewn in • Scar therapy | <p>F22 - Velcro Fastener Specify velcro crotch panel</p> |
|  | <p>Thorax Garment With Zipper</p> <p>Includes hook and eye closure under the flap.</p> <ul style="list-style-type: none"> • Hook & eye closure under the flap allow for easy zip closure • Zipper can be located in front or back | <ul style="list-style-type: none"> • Easier to put on and take off • Scar therapy | <p>Z01 - Zipper</p> |

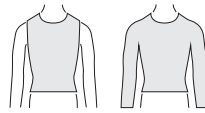



Custom Flat Knit Upper Extremity



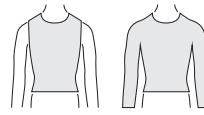


| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|---|---|
|  | <p>Thorax Garment With Hook and Eyelet</p> <ul style="list-style-type: none"> • Can be located in front or back | <ul style="list-style-type: none"> • Expansion possibility via second eyelet row • Edema therapy • Scar therapy | <p>F21 - Hook & Eyelet</p> |
|  | | | |
|  | <p>Thorax Garment With Velcro Closure</p> <ul style="list-style-type: none"> • Can be located in front or back | <ul style="list-style-type: none"> • Alternative closure option • Edema therapy • Scar therapy | <p>F22 - Velcro</p> |
|  | <p>Thorax Garment With Velcro to a Compression Stocking</p> | <ul style="list-style-type: none"> • Secure connection of upper and lower part garments • Edema therapy • Scar therapy | <p>F22 - Velcro</p> <p>Specify to velcro to stocking</p> |
|  | <p>Thorax Garment With a Silicone Border at the Waist and Hips</p> | <ul style="list-style-type: none"> • Prevention of sliding • Edema therapy • Scar therapy | <p>S10 - 5 cm Silicone Border</p> <p>Specify silicone at waist and hips</p> |
|  | <p>Thorax Garment With Knitted Seamless Breast Cups</p> <ul style="list-style-type: none"> • Breast cups are lightly compressive • Pockets for prosthesis are available | <ul style="list-style-type: none"> • Prevents abrasions and pressure points in the chest area • Edema therapy • Scar therapy | <p>B01 - Breast Cups</p> <p>B03 - Prosthesis, Breast Pockets</p> |

Custom Flat Knit Upper Extremity




| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|--|--|
|  | <p>Thorax Garment With Open Breast Area</p> | <ul style="list-style-type: none"> • Allows patient to wear combined with standard bra • Edema therapy • Scar therapy | <p>B02 - Open Breast Area</p> |
|  | <p>Thorax Garment With Arm Extension and Zipper</p> | <ul style="list-style-type: none"> • It is easier to put on/take off the sleeve • Edema therapy • Scar therapy | <p>Z01 - Zipper</p> |
|  | <p>Thorax Garment With Stand-up Collar</p> | <ul style="list-style-type: none"> • Higher compression in neck area • Scar therapy | <p>Please specify stand up collar</p> |
|  | <p>Thorax Garment With Non-elastic Back Area</p> <ul style="list-style-type: none"> • Recommended for infants and small children | <ul style="list-style-type: none"> • Counteracts posture • Scar therapy | <p>Please specify non-elastic back</p> |

Custom Flat Knit Upper Extremity

| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|---|--|---|
|  | <p>Thorax Garment With Zipper and Pull</p> | <ul style="list-style-type: none"> • Greater ease in opening and closing the zipper. • Edema therapy | <p>Z01 - Zipper</p> |
|  | <p>Thorax Garment With Fastener, Sleeveless</p> <p>Available with zipper, hook and eye or Velcro</p> | <ul style="list-style-type: none"> • Provides pressure in the abdominal region. • Not compressive in the chest area. • Scar therapy for the abdominal area. | <p>Z01 - Zipper F21 - Hook & Eye F22 - Velcro</p> |
|  | <p>Thorax Garment, Sleeveless</p> <p>Style: TH</p> <ul style="list-style-type: none"> • Slip-on style • Available sleeveless only • Requires patient mobility | <ul style="list-style-type: none"> • Smooth and inconspicuous underclothing • Provides pressure in the abdominal region. • Not compressive in the chest area. • Scar therapy | |

Custom Flat Knit Upper Extremity

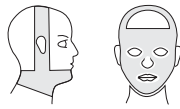

Compression Therapy for Facial Area


Lymphedema Therapy: Compression is applied at swell points in the face area, where lymphedema can occur. Compression is used after manual lymphedema drainage to retain the decongested state.

Burn Therapy: Compression Therapy in facial area can begin as soon as the wound closes and the patient is in stable condition. It is important when using compression therapy in the facial area to ensure that concave points by the eyes or nose are also cushioned with a pad, so that these specific areas can be treated with compression if needed.


Neck, Chin and Head Garments



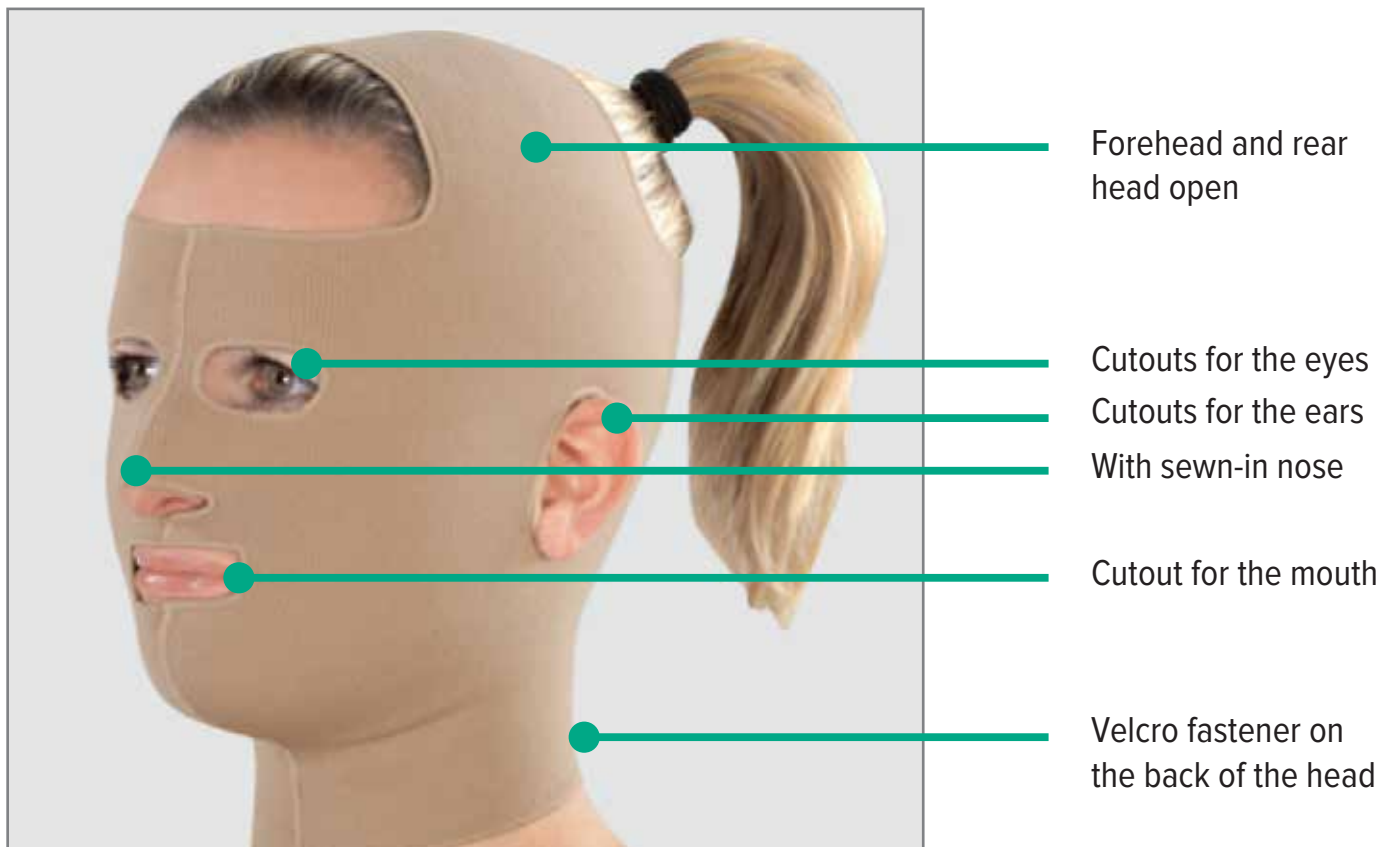


Before surgery Juzo garments can be individually customized according to the body measurements of the person to be treated so that the garment can be used for post procedure treatment. Juzo offers a variety of face masks for neck and head depending on treatment needs and extent of the scars.

Designed with all seams on the outside of the garment. Neck, chin and head garments are made in the Expert and Expert Silver fabrics.

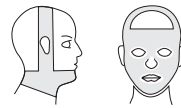

Custom Flat Knit Upper Extremity



| PRODUCT | STYLE | DESCRIPTION / USE | OPTION CODES |
|--|--|---|--|
|  | <p>Neck/chin Bandage</p> <ul style="list-style-type: none"> • Velcro or hook fastener on the back of the head and/or above the head band | <ul style="list-style-type: none"> • Secure placement of the chin area • Scar therapy • Edema therapy | <p>F21 - Hook & Eye F22 - Velcro</p> |
|  | <p>Face Mask With Forehead and Back Of The Head Closed</p> <ul style="list-style-type: none"> • Velcro or hook and eye fastener on the back of the head | <ul style="list-style-type: none"> • Complete head and face treatment • Cutouts for the eyes, nose, mouth and ears • Scar therapy • Edema therapy | <p>F21 - Hook & Eye F22 - Velcro</p> |
|  | <p>Face Mask With Forehead and Back of Head Closed and Face Area Open</p> <ul style="list-style-type: none"> • Velcro or hook and eye fastener on the back of the head | <ul style="list-style-type: none"> • Individual treatment possibility • Scar therapy • Edema therapy | <p>F21 - Hook & Eye F22 - Velcro</p> |
|  | <p>Face Mask With Forehead, Face and Back of Head Open</p> <ul style="list-style-type: none"> • Velcro or hook and eye fastener on the back of the head | <ul style="list-style-type: none"> • Individual treatment possibility • Scar therapy • Edema therapy | <p>F21 - Hook & Eye F22 - Velcro</p> |
|  | <p>Face Mask With Forehead and Back of Head Open And Sewn-in Nose Piece</p> <ul style="list-style-type: none"> • Velcro or hook and eye fastener on the back of the head | <ul style="list-style-type: none"> • Head compression treatment including nose • Scar therapy • Edema therapy | <p>F21 - Hook & Eye F22 - Velcro</p> |

Made to Measure Online Ordering

Trouble-free orders and re-orders through ShopJuzo online made to measure ordering, you can select the appropriate garment information, enter your patient's measurements, plus add notes and more.

System features that simplify the process for you include:


Faster Delivery. Direct transfer of measurements means quicker production and shipments.

Order Confirmation. You'll receive an email confirming and detailing your order.

Measurement Notifications. If you forget to enter a measurement or a measurement falls outside of the tolerance range, the system will notify you.

Customer Library. The system stores previously entered measurements and garment information for every patient to make future orders a breeze.

shopjuzo.com

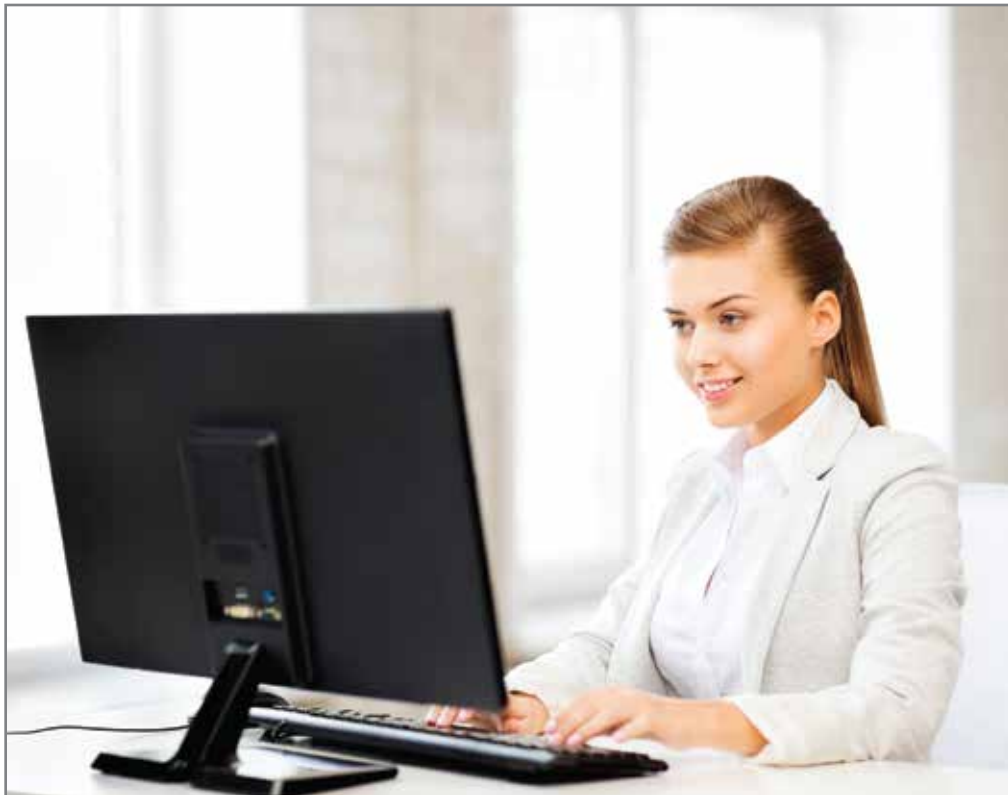

Juzo USA
P.O. Box 1088
Cuyahoga Falls, OH 44223
www.juzousa.com
support@juzousa.com

Juzo Canada Ltd.
1100 Burloak Dr. Ste. 300
Burlington, ON L7L 6B2
www.juzo.ca
support@juzo.ca

© Juzo, Keep Moving, Fiber Soft and Slippe Gator are registered trademarks of Julius Zorn, Inc. Lycra is a registered trademark of Invista Inc. Tyvek is a registered trademark of Dupont. © 2018 Julius Zorn, Inc.