

Comprefit®

Strongest Containment



Compreflex®

Easy & Measurable



Coolflex™

Most Breathable

Inelastic Wraps Guide

LOWER EXTREMITY



Simple Solutions for

Venous Disease, Lymphedema & Wound Care

Compreflex Standard

CALF & FOOT

20-50mmHg





Garment Features

- 20–50mmHg of adjustable resting compression accurately measured using our patented Accutab® Device
- Stretch panel secures garment for easy donning
- Conforming Breathe-O-Prene® provides a comfortable fit
- Patients outside of the standard product size range can use the Compreflex Strap Extender for an additional 10cm in circumferential length
- Billable under Medicare Code A6545-AW for patients with active venous stasis ulcers

Indications

- · Lymphedema
- · Edema
- · Chronic venous disease (CVD)
- Chronic venous insufficiency (CVI) including, but not limited to: ulcer due to venous stasis
- · Lipedema

When to Use

- · Patient requires a calf and foot garment
- Varying ranges of adjustable resting compression is desired (20–50mmHg)
- · Non-attached foot piece is desired
- · No compression in the liner is desired

Ideal for Patients Experiencing

- · Active Venous Stasis Ulcers
- · Obesity
- Poor Skin Condition
- Fibrotic Tissue
- · Irregular Shaped Limb
- · Poor Grip Strength
- · Unable to don traditional Compression Stockings





Pair of Basic Liners

Non-Compressive







Wrap Details

Variable Compression

Each Compreflex Calf wrap includes the Accutab, allowing users to accurately adjust the garment to the desired level of compression

Intelligent Design

Garment features a contoured calf, providing a superior fit

Premium Materials

Soft, conforming Breathe-O-Prene provides a comfortable fit

Superior Quality

Each garment is double stitched to ensure durability and is backed by a six-month warranty*



Comprefit Standard

CALF & FOOT

30-40mmHg





Garment Features

- · 30-40mmHg firm, non-stretch compression
- Rigid, non-stretch garment profile providing a high level of containment
- · Two slot options provide a comfortable fit
- Garment can be adjusted to include an additional 10cm of circumferential range by adding the Comprefit Standard Strap Extenders

Indications

- Lymphedema
- · Edema
- · Chronic venous disease (CVD)
- Chronic venous insufficiency (CVI) including, but not limited to: ulcer due to venous stasis
- · Lipedema

When to Use

- · Patient requires a garment with no stretch
- Patient requires compression in foot; product includes Compreboot
- · A high level of containment is needed
- · No compression in the liner is desired

Ideal for Patients Experiencing

- · Active Venous Stasis Ulcers
- · Obesity
- · Poor Skin Condition
- · Irregular Shaped Limb
- Unable to don traditional Compression Stockings





Wrap Details

Variable Compression

Comprefit is ideal for patients with chronic venous disease or those needing compression therapy following a vascular procedure.

Optimum Fit

Select the slot that provides the most conforming fit

Easy To Use

Strap ends close to prevent tangling

Inelastic Compression

Provides active, gradient compression for reduction of edema



Adjustability

Garment straps may be trimmed as patient reduces in size. Strap Extenders may be used to gain 10cm of circumferential range



Comprefit is pictured with the Compreboot.

Compreboot is sold separately

Coolflex Standard

CALF

20-40mmHg





Garment Features

- · 20-40mmHg of lightweight, gradient compression
- Garment is made of Spacer Fabric TR™ and includes superior properties preventing the trapping of heat and promoting the transfer of moisture
- EZ Close Pressure™ system visually notes the compression levels of 20-30 and 30-40mmHg on the garment straps for the easy application of compression
- · Hook closures for maximum strength and durability
- One-handed side closure system promoting independence and patient adherence

Indications

- · Lymphedema
- · Edema
- · Chronic venous disease (CVD)
- Chronic venous insufficiency (CVI) including, but not limited to: ulcer due to venous stasis
- · Lipedema

When to Use

- · Patient requires a lightweight material due to skin sensitivities or fragility
- Varying ranges of adjustable resting compression is desired (20–40mmHg)
- · Patient has limited hand strength or dexterity
- Ability to wear standard footwear over liner is desired
- Patient has difficulty leaning forward or bending, lateral or side closure to aid the patient in applying the garment is needed

Ideal for Patients Experiencing

- · Recurring venous stasis ulcers
- · Fragile or broken skin
- · Skin sensitivities
- · Poor hand strength
- · Obesity
- · Inability to don traditional compression stockings









Wrap Details

Variable Compression

Coolflex is a lightweight compression device optimal for individuals with fragile or broken skin. Its patented EZ-Close pressure system closes laterally which can promotes independence and improves patient outcomes.

Most Breathable

The Spacer Fabric TR is designed to be highly breathable for cool-wearing comfort

One-Handed Donning

EZ-Close Pressure System of eight tabs makes one-handed, lateral donning simple, even for patients with dexterity issues

Innovative Design

Durable, metal hook closures for maximum strength and durability



Accurate compression is easy with Coolflex Standard Calf

The integrated EZ-Close Pressure System provides a visual indicators for 20–30mmHg and 30–40mmHg on each tab.





Please consult your doctor, or other qualified health care professional before using any product discussed within the literature. See product insert for complete information.

* Garment includes a 6-month warranty against defects in materials and workmanship